

CYNOSURE®

Vivace™

Fine lines and wrinkles
General skin tightening and toning





VIVACE

Fractional RF Microneedling

BEFORE AND AFTER



Before



After 3 TX



Before



After 3 TX



Before



After 3 TX



Before



After 3 TX

FREQUENTLY ASKED QUESTIONS

What does VIVACE feel like?

Your clinician will apply numbing cream to your face to increase comfort level. You will experience a light pressure to the face.

Does VIVACE hurt?

The numbing cream is applied 30 to 45 minutes prior to the treatment to deliver a very comfortable patient experience. Some patients described the treatment to be in a range of comfortable to virtually pain-free.

How do I determine the right treatment for me?

It's important to work with a trusted clinician who uses a FDA cleared and TGA listed device to give you confidence in its safety, reliability and efficacy.

What is RF?

Radio frequency provides heat that stimulates collagen production, when combined with state-of-the-art Microneedling, patients have experienced optimal results for overall skin tightening, fine lines and wrinkles, improvement in scars, spider veins, stretch marks, pore reduction and improved skin tone and texture.

How many treatments do I need?

Many clinicians find three treatments, spaced four to six weeks apart, provide the best #vivaceskin experience. Results can take up to three months to fully appear as the process of collagen regeneration increases over time.

How long does it take?

The actual treatment takes about 45 minutes, with an additional 30 to 45 minutes needed for numbing.

When can I wear makeup?

Most patients go out hours after the treatment with no makeup at all. We recommend at least six hours before you apply makeup, or you may even wait until the next morning.

How do I get the maximum result from my treatment?

Follow the recommended recovery protocols once you get home. Avoid the sun immediately after and abstain from any heavy work out over the next 24-48 hours. Multiple treatments are recommended for optimal results.

What kind of skin does it work on?

Most skin colours can be treated, including tanned skin and darker skin. (Fitzpatrick skin types I-VI)

How long does the result last?

Everyone is different, but the collagen you make is your own and production of collagen continues over the next three months. Results will improve over time.

What is my recovery time?

The clear, microchannels created by the Vivace remain open for four to six hours – unique to the Vivace. After this time, you are ready for a night out, with very little to no indication of any treatment that day.

- ◎ Minimal downtime
- ◎ Comfortable treatment
- ◎ Great results!

Vivace™

#VIVACESKIN



GET YOUR CONFIDENCE BACK!

Ask us for the VIVACE
treatment plan that is right for you

 Cynosure Australia  Cynosure Australia #VIVACESKIN

Cynosure (Australia) Pty Ltd
14-16 Suakin St,
Pymble NSW 2073
+61 2 9484 4546 | infoaustralia@cynosure.com

cynosureaustralia.com

© 2021 Cynosure, Inc. ALL RIGHTS RESERVED. Cynosure Australia is the exclusive distributor for the marketing, sales and delivery of VIVACE in AUS | NZ. Cynosure is a registered trademark of Cynosure, Inc.

CYNO}SURE®